

**TekMed™**

1300 720 727

[www.tekmed.com.au](http://www.tekmed.com.au)

**TekMed NZ™**

0800 456 217

[www.tekmed.co.nz](http://www.tekmed.co.nz)



FS521104



**EMODIAL**

**EXIT-POCKET® WP**



**EMODIAL**

Via Finati, 4/A - Ferrara - ITALY

Tel. +39 0532 978198 - Fax +39 0532 978414

[www.emodial.com](http://www.emodial.com) - [info@emodial.com](mailto:info@emodial.com)



Ref. GB0013H - Rev. 06/2019

**Waterproof Pocket  
for Catheter Protection**

# EXIT-POCKET® WP

### Description

**EXIT-POCKET WP** is a waterproof pocket closed on four sides provided with adhesive hole to be fixed to the skin.

### Indications


Containment of catheter lumens during patient's hygiene procedures (shower and bath) and during water activities (diving, sea, swimming pool).

### Purpose

Designed to protect the exit-site from water contact. The device is completely waterproof: if well-positioned it protects the exit-site from contact with water preventing the possibility of microbial contamination.

### Benefits

- Quick and easy placement.
- Comfortable fit.
- Helps to improve the quality of life of catheterized patients.
- Easy and fast removal.

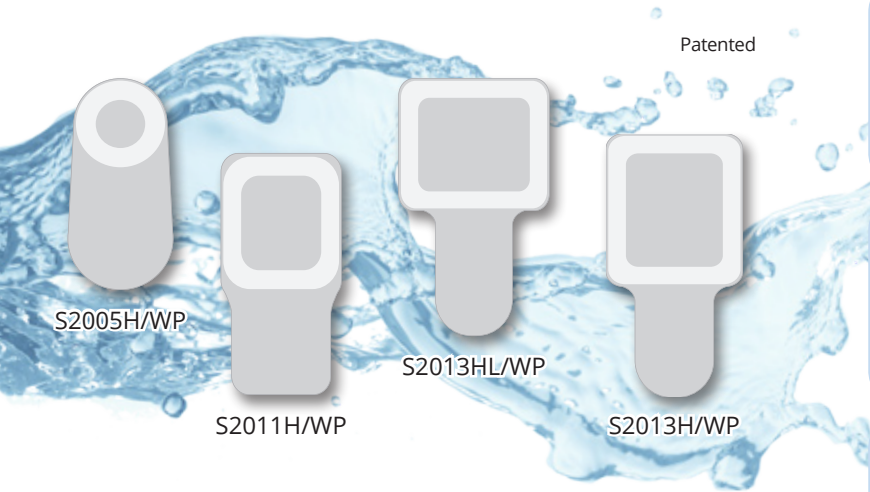

### References

Data and technical specifications can be changed without notice.

CODE	SIZE (cm)			INDICATION	PCS / BOX
	Hole				
S2005H/WP	ø 5	11	22	Bath and Shower	50
S2011H/WP	10 x 9	13	26	Bath and Shower	50
S2013HL/WP	13 x 11	17,6	30	Bath and Shower	50
S2013H/WP	11 x 13	15,6	30	Bath and Shower	50

## Waterproof pocket for catheter protection

### Directions for use

1. Cover the catheter lumen(s) with the device.
2. Remove the protective sheet from the adhesive.
3. Place the lower margin of the adhesive hole in correspondence to the lower part of the exit-site. Complete the fixing from bottom to top, avoiding wrinkles or airlocks .
4. Take shower or bath paying attention not to rub the area near the adhesive.
5. Remove gently from the skin a small strip of adhesive starting from the lower part. Pull carefully with small movements from bottom to top.



CLASSIFICATION	• CLASS I MEDICAL DEVICE (EUROPEAN COMMUNITY)
BIOCOMPATIBLE	• ACCORDING TO UNI EN ISO 10993 STANDARDS
STERILIZATION	• NOT STERILE
EXPIRATION	• 2 YEARS
	• SINGLE-USE