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INSTRUCTIONS FOR USE

MODE D'EMPLOI

**TekMed™**  
1300 720 727  
www.tekmed.com.au

**TekMed NZ™**  
0800 456 217  
www.tekmed.co.nz

FS521104

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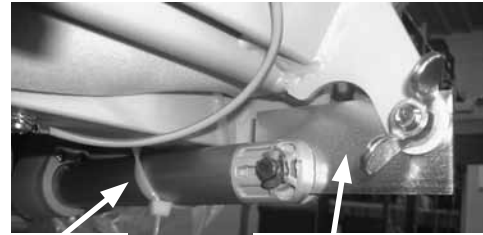
Abb. / Fig. 1

## A. Unpacking

**Use a forklift for transportation! Do not cut into the carton!**

- Remove outer straps from pallet
- Lift carton and put it aside
- Remove inner straps
- Remove the chair from pallet base

**Do only use the flat oval pipes of the base frame to carry the chair.**

## B. Assembly

### 1. Remove transport safety device

**Remove the transport safety device and cable tie before using the chair! (Abb. / Fig. 1)**

After the transportation safety device and the cable tie have been removed (Abb. / Fig. 2), connect the push rod of the leg segment motor with the receiving strap on the bottom side of the leg seg-

ment. Now fix rod and leg segment with the bolt and the circlip. (Abb. / Fig. 3)

### 2. Inserting the head segment

Insert the oval tube of the head segment in the appropriate aperture in the rear of the back segment. The height of the head segment is variable. Choose the appropriate setting and then fix the head segment with the screw.

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### 3. Inserting the armrests

Insert the armrests on the back shaft. Hold armrest in vertical position and turn the armrest with slight counter-pressure to achieve locking by the slide catch. (Abb. / Fig. 4) Once the armrest is fully inserted, it can be moved up- and backwards as desired.

**Make sure that the armrest cannot be removed in horizontal position. Improper mounting can damage the armrest.**

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### 4. Electrical outlet

#### Connect the therapy chair with the power supply system.

Passing over the power cord with casters should be avoided by appropriate installation. To avoid any damage, the net cable has to be rolled up before moving the chair and has to be laid on the seat or hung up over the arm-rest in the rolled up condition.

The ComfortLine with rechargeable battery is only run by the integrated rechargeable battery. Therefore it does not have any power cord.

### 5. On/Off switch

The On/Off switch is located on the right side under the seat surface at base structure level. The mains connection should be switched off if the dialysis chair is not in use. After switching off please ensure that all buttons of the remote control are locked. (Abb. / Fig. 7)

The ComfortLine with rechargeable battery has the on/off switch on the upper side of the rechargeable battery. To switch off the therapy chair press the red rotating switch. By rotating and lifting

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the switch the ComfortLine with rechargeable battery becomes activated and the motors can be run.

(Abb. / Fig. 5 and Abb. / Fig. 6)

## 6. Installing the seat section

Clamps are provided on the bottom side of the seat section. Use these clamps to fix the seat section in the corresponding recesses in the frame. Therefore it is necessary to adjust the back segment backwards. (Abb. / Fig. 8)

**Please make sure that the notches of the seat segment are locked and secured with the three notches on the corresponding bolts of the seat frame. If not locked properly, the seat segment might slide unintentionally.**

## 7. Equipotential bonding

ComfortLine includes an equipotential bonding as standard. The connection bolt according to the DIN 42801 standard is located on the right back of the base frame.

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## C. Instructions for handling

### 1. General

Intended use for Bionic therapy chairs is the temporary position of patients or donors during treatment under medical supervision. By using the chair for x-ray diagnostics please consider that it is opaque to radiation.

The chair does probably not meet the needs of (especially mentally) handicapped people. In case of doubts medically skilled staff should judge whether the chair can be used or not.

The chair is designed for people of normal size. It can be adjusted to the patient size by pulling out the head rest and/or adjusting the footrest. When not using the chair it should be moved into the lowest position to avoid the risk of injuries when getting in or out of the chair.

**!** All users of the chair, especially personnel, patients or donors have to be instructed in the handling of the chair and have to be made aware of the risks. We recommend to hand out this instruction to all users and to make sure that it is read and well understood.

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**!** Please make sure that movements of the chair are not restricted by architectural facts i.e. window sills, maintenance cables etc. Damage of the back segment motor or of the joints can occur. (Abb. / Fig. 9)

If the chair is used in combination with a table, height adjustment may only be engaged when the table is removed.

The ground clearance of the chair is 110 mm, in case of ComfortLine with low entrance only 30 mm.

### 2. Functions of remote control

The ComfortLine is equipped with 5 motors for the adjustment of the different position by hand switch. (Table)

### 3. Shock position

The shock position is set using the foot switch at the front of the base structure. Therefore the motors work at higher speed and this is why the foot pedal may not be used for normal adjustment of the chair. (Abb. / Fig. 10)

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#### 4. Removing the armrest

In order to facilitate the entrance from a wheelchair into the therapy chair, the armrests can be removed. In case of ComfortLine with low entrance it is possible to adjust the therapy chair to an entrance height of 50 cm.

For this it is necessary to swing the armrest backwards to reach the slide catch and the armrest can be pulled off the chair.

#### 5. Adjustment of standard armrests

Underneath the ball and socket joints of each armrest is a locking lever. In the unlocked setting, the armrest moves freely around the ball for adjustment. Turn the armrest into the standard position in a right angle to the backrest and parallel to the sitting area. For locking the desired position, the locking lever is to be tightened clockwise with moderate force. The friction in the ball and socket joint will easily hold the weight of the arm.

**Do not move the armrest without releasing the locking lever.**

If the locking lever is pulled downwards, it can be rotated freely and adjusted to a parallel position. When pulled back by the re-

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lease spring, the lever is engaged with the screw for locking and unlocking (Abb. / Fig. 11).

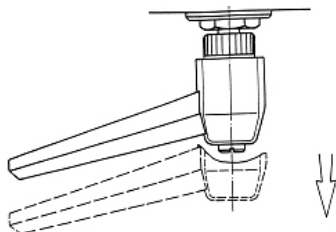
Abb. / Fig. 11

It is also possible to lower the armrest by approx. 20° from horizontal position. The chair is always delivered with armrests in horizontal-blocked position. To loosen this fixation the bolt has to be pulled outwards while the locking lever is released. Afterwards the locking lever should be fixed (Abb. / Fig. 12).

**Before first use of this function the screw as well as the bolt have to be loosened.**

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#### 6. Adjustment of transfusion armrest (optional)

In the basic position both armrests are located parallel to the seat.

By unscrewing the first locking screw the armrest can be moved freely. In order to fix the adjusted position, tighten the locking screw clockwise with moderate force. Because of the frictional engagement in the determined ball joint the normal weight load of the patient's arm will not change the position of the armrest (Abb. / Fig. 14).

The armrest upholstery can be adjusted forwards and backwards by unscrewing the knurled knob. As the desired position is adjusted, this position can be fixed by tightening the knurled knob (Abb. / Fig. 15).

The second locking screw is necessary to change the height of the armrest. After the desired position is adjusted, it can be fixed by tightening the locking screw (Abb. / Fig. 13).

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Abb. / Fig. 15

**7. Adjustment of heightadjustable armrest (optional)**  
 With the optional height adjustable armrest the elbow rest height can be altered. Lift the back upper end of the armrest upholstery to select one of three positions where it will snap into position by its own weight. For returning it to the lowest position, it is to be pulled up to the limit stop until the snap mechanism will disengage.

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**8. Adjustment of footrest**

To remove the footrest, pull the plastic part of the footrest receptacle gently; tilt the footrest into the reverse direction and take off the footrest.

By reversing the footrest can be adjusted to small and large patients. Precise adjustment of the footrest is performed by the electrical motor operated by the hand switch. (Abb. / Fig. 16)

**9. Height adjustment**

The therapy chair can be height adjusted by hand switch. During dialysis the lowest position should be used for safety reason.

**10. Central fixing brake**

The chair has a central fixing brake to block four wheels. To activate the brake, the pedal at the base frame on the right-hand side of the dialysis chair is pressed down or on the left pressed upward. The brake is deactivated by pressing in opposite direction.

**11. Entry and exit**

The patient should enter and exit from the side while the central brake is fixed. Both armrests should be swung backwards.

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**The footrest is not intended to be used as a step. Stepping on it with full body weight can tip over the chair.**

**The armrests are not intended to be used for the entry. If the armrest is stressed excessively especially in the front it could break. The maximum load for each armrest is 40 kg.**

Leaning or sitting of the patient or a second person on the backrest or the leg segment in horizontal position as well as on the armrests is inadmissible.

Extreme dynamic stress from falling back or rocking may also overload the construction.

### 12. Electrical safety

The therapy chair complies with the electrical requirements to EN 60601, IPX4, class II and type B.

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### 13. Battery operation in case of power failure

The chair has two exchangeable 9-Volt dry cell batteries at the power supply. In case of power failure, the battery supplies power to return the chair. The battery has capacity for one adjustment and should be replaced after an emergency operation.

Regarding the ComfortLine with rechargeable battery there are no exchangeable 9-Volt dry cell batteries.

### 14. Accessories the chair can be equipped with

- Seat cushion with visco-elastic foam (we recommend exchanging the visco-elastic foam core every 3 years to guarantee the sitting comfort)
- Seat cushion with spring core
- Handle on the back
  - with Reading lamp (in case of the ComfortLine with rechargeable battery it has its own separate power plug)
  - with Paper roll holder
- Protecting cover for leg and foot rest
- Height adjustable armrests
- Transfusion armrests
- Receptacle for console table or IV-pole

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- Console table
- IV-pole
- Velo Exercise Trainer

Assembling of accessories only by authorised personnel. All accessories should only be used in combination with the ComfortLine.

#### 15. Reading lamp (optional)

The reading lamp is delivered exclusively in conjunction with handle on the back segment. It is switched on and off using the handswitch.

The reading lamp may not lie completely on the upholstery, otherwise damages of the upholstery can occur. The minimum distance is 30 cm.

In case of the ComfortLine with rechargeable battery the lamp has its own separate power plug and on/off switch. It cannot be operated by the hand switch of the chair.

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#### 16. Rechargeable battery operation (optional)

Before starting the first time it is recommended to charge the battery completely over a whole night. The capacity of a new and completely charged battery is under normal conditions adequate for approximately a whole week.

**While non-operating the battery has to be recharged every 6 weeks.**

It is recommended to recharge the battery regularly. In this case there is no risk of losing durability because the rechargeable battery knows no memory-effect and is secured against overcharging.

An acoustic signal shows that the battery needs to be recharged directly. At this time the rechargeable battery has still enough power to bring the patient into a safe position. Afterwards the battery needs to be recharged directly over a period of 10 to 12 hours. Therefore the battery can be taken out of the attachment or the therapy chair itself can be moved to a power socket. The

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battery is charged with the help of the associated battery charger. The green light shows that the battery and the charger are correctly connected. It is recommended to charge the battery at least one hour / or until the yellow light turns out. Although the partly charged battery offers enough capacity for some changes of load we recommend charging the battery always completely. The charge time of an empty battery is approx. 10 to 12 hours.

If the rechargeable battery is stressed further after the acoustic signal is hearable the motors get very slow. Besides it can happen that functions which need to drive more than one motor (shock position / zero-position) do not work properly. If all these warnings are ignored and the battery is stressed further it could lead to a damage of the rechargeable battery.

Also it is recommended to replace the rechargeable battery every 5 years.

 **It is only allowed to use the provided battery charger.**

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#### 17. Velo exercise trainer (optional)

To mount the Velo remove the footrest, as already described. Then mount the Velo as shown (Abb. / Fig. 17).

 **Touch the Velo only on the tube bend, pedal arms can heat up.**

 **The Velo is not intended to be used as a step. Otherwise the chair can tip over!**

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## D. Maintenance, Inspection and Care

We propose an annual inspection of the chair. A safety-related check (measurement of the patient leakage current according EN VDE 0751) is also possible.

We recommend inclusion of the following points in the inspection:

### External damage

- Damage to casings (hand switch, motor casing, mains plug)
- Damage to cables (insulation, pressure and pinch points)
- Function of cables' strain relief
- Damage to plug contacts (bend pins)
- Damage to frame parts (deformation, wear and tear)
- Damage to accessories connected to the product (deformation, wear and tear)
- Damage to O-rings (if present) of plug contacts
- Function of the mains plug
- Function of ball and joint socket of armrests (screwing)

### Functional examination

First switch all drives to end position. When the end position is

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reached the drives have to stop.

Check the operation of the traction relief in the sites where cables emerge from the casings of the chair.

Batteries should be replaced at least once a year in drives with emergency power units.

Also Teflon® spray should be used annually for the activate control of the footrest, the armrest tubes, the back segment arbor and the connection of the back segment arbor to guarantee a smooth mobility.

The Chair must not be cleaned with a „Jet-Stream“ washing machine or a pressure washer. We recommend a wet cloth and for disinfection a wipe disinfection. The upholstery is largely resilient to many alcohol-based cleaning and disinfection agents, but also to mild soaps. In case of doubt, a test on a small surface outside the visible area is advisable.

**Strong acidic or alkaline disinfectants or disinfectants comprising aggressive substances (e.g. peroxide) can cause irreversible damage to the chair materials.**

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06172 - 7576 - 63

## E. Type plate

The type plate with serial number is attached to the base frame. For technical questions and spare part orders please indicate type of chair and serial number. (Abb. / Fig. 18 and 19)

For technical information, documentation, service and spare parts please contact directly:

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## F. Disposal

### 1. Packaging

The pallet is single use only and may be used for other purposes or can be incinerated.

The outer box is corrugated carton. It may be recycled or incinerated. Please follow appropriate procedure for waste disposal.

### 2. Therapy chair

For disposal the local regulations should be observed.

Bionic Medizintechnik GmbH ensures that therapy chairs, which were delivered after 13.08.2005 and covered by the WEEE, be disposed of in accordance with the directive of the Electrical.

Because the therapy chairs are medical equipment in accordance with § 2 para 1 no. 8 it deals with possible hazards from contamination. Therefore the following conditions for withdrawal and the free disposal shall apply:

- Professional, documented decontamination of therapy furniture and verification of this to Bionic

- Authorize delivery by Bionic
- Free delivery to the site Friedrichsdorf

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## G. Troubleshooting

Fault	Cause	Action / Repair
Therapy chair does not move	Mains voltage is missing Mains error	Check fuse Check mains voltage Replace batteries Check on/off switch
	Mains voltage is missing and battery is discharged	Insert mains plug Replace batteries
	Power supply is switched off	Switch on main switch
	Plugs' connections are not correct	Check all plugs and reconnect if necessary
	Fuse on board is damaged	Replace fuse
	Power supply is damaged	Replace power supply
Therapy chair moves slowly and then stops	Mains voltage is missing and chair is running by battery and battery is discharged	Check mains voltage Insert mains plug Replace battery
One motor interrupts	Motor end switch is damaged	Replace motor

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Fault	Cause	Action / Repair
One motor does not move	Remote control is damaged	Check if remote control of another chair is working, if necessary replace remote control
	Plug is not connected or motor cable is damaged	Check all plugs, insert plug or replace motor
	Motor is damaged	Check motor by connecting the plug to another socket, if necessary replace motor
Green LED light of remote control does not work or is always on	DIN plug is not plugged in or remote control is damaged	Insert DIN plug or replace remote control
Reading lamp (optional) does not work	LED is damaged	Replace LED
	Mains plug is not connected	Connect plug
	Power supply is damaged	Replace power supply or adapter for control
	Y-cable of reading lamp is not connected or damaged	Check connection and replace cable if necessary
Foot pedal for shock position does not work	No contact	Adjust all screws underneath the foot pedal

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Bionic Medizintechnik GmbH ... a JMS-corporation · Max-Planck-Straße 21 · D-61381 Friedrichsdorf/Germany